



TESTING KITS FOR DRUGS

# Procedure Card for IDT0150

## BARBITURATES IDENTIFICATION

### Substance Test



**Read  
Before  
Use**

As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

<b>Test Preparation</b>	<b>1</b>	Remove the Smart-Tip™ probe from test unit.	<b>2</b> Sample suspected substance.	
			<b>Powder</b>	<b>Tablet</b>
			Touch Smart-Tip™ probe to sample. 	Firmly press Smart-Tip™ probe on the tablet while scrubbing in a circular motion until the tip is completely covered with substance. 

<b>Test Preparation</b>	<b>3</b>	Lock Smart-Tip™ probe into test unit. Press firmly.	<b>STOP</b> <b>Do not continue unless probe is locked below the Flex Pins and cannot be removed from the test unit.</b>	<b>Identification</b>	<b>4</b>	With <b>thumb and forefinger</b> , squeeze chamber 1 until capsule breaks.	<b>5</b>	Gently shake test unit for 2-3 seconds.

<b>Identification</b>	<b>6</b>	With <b>thumb and forefinger</b> , squeeze chamber 2 until capsule breaks.	<b>7</b>	Gently shake test unit for 2-3 seconds.

**Results**



The appearance of an intense violet color indicates the presence of **Barbiturates**.



If no intense violet color develops, Barbiturates are not present.



**Disposal**

The test kit is safe to dispose of in the trash.

01-19



World Leader in Forensic Products

Manufactured by IDenta Ltd

www.identa-corp.com

Patented Technology

Made in Israel