



DRUG PRECURSOR TESTING KITS


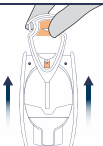

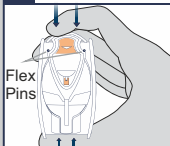
Procedure Card for IDT0280T


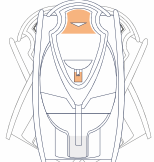
GBL IDENTIFICATION

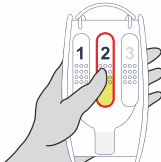
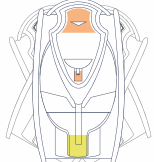
Substance Test




Read 
Before Use

As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

Test Preparation	1	Remove the Smart-Tip™ probe from test unit.	2	Sample suspected substance. Liquid Place the absorbent paper of Smart-Tip™ probe into liquid. Make sure paper is completely wet.	3	Lock Smart-Tip™ probe into test unit. Press firmly.	 Do not continue unless probe is locked below the Flex Pins and cannot be removed from the test unit.
							

Identification	4	With thumb and forefinger , squeeze chamber 1 until capsule breaks.	5	Gently shake test unit for 2-3 seconds.
				

Identification	6	With thumb and forefinger , squeeze chamber 2 until capsule breaks.	7	Gently shake test unit for 2-3 seconds.
				

Results		The appearance of a light orange color indicates the presence of GBL (γ -Butyrolactone).
	 	If a light orange color does not develop or the liquid remains a yellow color, GBL is not present.



Disposal

The test kit is safe to dispose of in the trash.

01-19



World Leader in
Forensic Products

Manufactured by IDenta Ltd

www.identa-corp.com

Patented Technology

Made in Israel