



TESTING KITS FOR DRUGS

Procedure Card for IDT0550

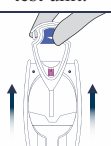


PCP IDENTIFICATION

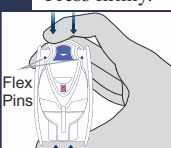


Substance Test

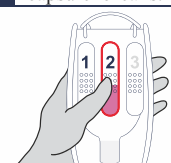
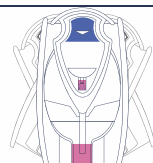




**Read
Before
Use**

As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

| | | | | |
|------------------|---|---|--|--|
| Test Preparation | 1 | Remove the Smart-Tip™ probe from test unit. | 2 Sample suspected substance. | |
| |  | | Substance Touch Smart-Tip™ probe to sample.  | Tablet Firmly press Smart-Tip™ probe on the tablet while scrubbing in a circular motion until the tip is completely covered with substance.  |

| | | | | | | | | |
|------------------|--|--|---|----------------|--|--|---|--|
| Test Preparation | 3 | Lock Smart-Tip™ probe into test unit. Press firmly. | STOP Do not continue unless probe is locked below the Flex Pins and cannot be removed from the test unit. | Identification | 4 | With thumb and forefinger , squeeze chamber 1 until capsule breaks. | 5 | Gently shake test unit for 10 seconds. |
| |  |  | | |  | | | |

| | | | | |
|----------------|---|---|---|--|
| Identification | 6 | With thumb and forefinger , squeeze chamber 2 until capsule breaks. | 7 | Gently shake test unit for 10 seconds. |
| |  |  | | |

| | | |
|---------|---|--|
| Results |  | The appearance of blue color particles adhering to the sampler tip or precipitating out of solution indicates the presence of PCP (Phencyclidine). |
| | | * Note: The liquid will remain dark pink. |
| |  | If no blue color develops, PCP is not present. |



Disposal

The test kit is safe to dispose of in the trash.

01-19



Manufactured by IDenta Ltd

www.identa-corp.com

Patented Technology

Made in Israel