

CUSPER™

TESTING KITS FOR DRUGS

Procedure Card for IDT0670

PHENETHYLAMINE

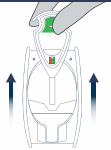
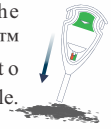
(2C)




IDENTIFICATION













Substance Test

Read Before Use

As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Once a capsule is broken, do not resqueeze the chamber or remove the probe. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

Test Preparation	1	Remove the Smart-Tip™ probe from the test unit.	2 Sample the suspected substance.	
			Powder	Tablet
		Touch the Smart-Tip™ probe to the sample.		

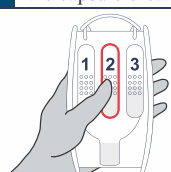

Test Preparation	3	Lock the Smart-Tip™ probe in the test unit. Press firmly.	STOP	Identification	4	With thumb and forefinger , squeeze chamber 1 until the capsule breaks.	5	Gently shake the test unit for 2-3 seconds.
					<p>Do not continue unless the probe is locked in place below the Flex Pins.</p>			



Results		The appearance of one of the colors below indicates the presence of Phenylethylamine (2C) .	
		Yellow changing to green	2C-B
		Yellow changing to light green	2C-C
		Yellow-brown changing to reddish brown	2C-D
		Dark Brown	2C-E
		Dark Green	2C-H
		Green-blue with yellow edges	2C-I
		Yellow-brown changing to rusty brown	2C-P
		Dark brown with green edges	2C-N
		Black/dark purple changing to orange-red	2C-T-2
		Black/dark purple changing to reddish-orange	2C-T-4
		Black/dark purple	2C-T-7



If none of the above colors develop, **Phenylethylamine (2C)** is not present.

Proceed to Dilution (Steps 6-9), regardless of whether or not a color appeared.

Dilution	6	With thumb and forefinger , squeeze chamber 2 until the capsule breaks.	7	Gently shake the test unit for 2-3 seconds.
				The test window will warm up. This is normal.

Dilution	8	With thumb and forefinger , squeeze chamber 3 until the capsule breaks.	9	Gently shake the test unit for 2-3 seconds.
				The test window will warm up. This is normal.



Disposal

After Dilution, the test kit is safe to dispose of in the trash.

ENG 07-22

IDenta
World Leader in Forensic Products

Manufactured by IDenta Ltd
www.identa-corp.com
Patented Technology
Made in Israel