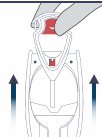

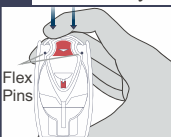
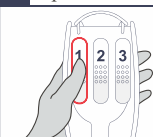











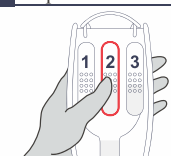

Read Before Use


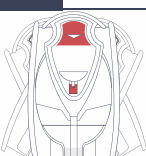
As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

Test Preparation	1	Remove the Smart-Tip™ probe from test unit.	2	Sample suspected substance.
			Powder	Touch Smart-Tip™ probe to sample in different places on the sample. Make sure the tip is completely covered. 

Test Preparation	3	Lock Smart-Tip™ probe into test unit. Press firmly.	STOP Do not continue unless probe is locked below the Flex Pins and cannot be removed from the test unit.	Identification	4	With thumb and forefinger, squeeze chamber 1 until capsule breaks.	5	Gently shake test unit for 2-3 seconds.
								

Results		The appearance of a red-purple (wine) color indicates the presence of Fentanyl .
		The appearance of a purple color <u>within one minute</u> indicates the presence of an opiate such as Heroin, Morphine or Codeine . * This does not exclude the possibility that Fentanyl may be present.
		The appearance of a reddish-orange color <u>within one minute</u> indicates the presence of Amphetamine or Methamphetamine . * This does not exclude the possibility that Fentanyl may be present.
		If none of the above colors develop, the above mentioned substances are <u>not</u> present in the tested sample.
 Proceed to Dilution (steps 6-9), regardless of whether or not a color appeared. 		

Dilution	6	With thumb and forefinger, squeeze chamber 2 until capsule breaks.	7	Gently shake test unit for 2-3 seconds.
				Wait 5 minutes. The test window will warm up. This is normal.

Dilution	8	With thumb and forefinger, squeeze chamber 3 until capsule breaks.	9	Gently shake test unit for 2-3 seconds.
				The test window will warm up. This is normal.



Disposal

The test kit is safe to dispose of in the trash.

06/18