

CUSPER™

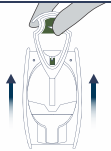


TESTING KITS FOR DRUGS

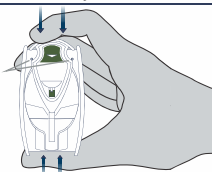
Procedure Card for IDT0840

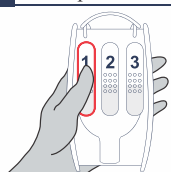
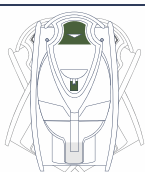
MECKE REAGENT IDENTIFICATION Substance Test





Read Before Use

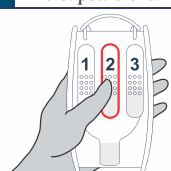
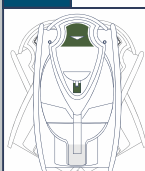
As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.


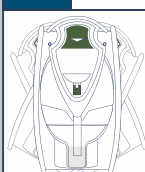
Test Preparation	1	Remove the Smart-Tip™ probe from the test unit.	2	Sample the suspected substance.			
				Powder	Tablet		
				Touch the Smart-Tip™ probe to the sample.		Firmly press the Smart-Tip™ probe on the tablet while scrubbing in a circular motion. Make sure to completely cover the tip with the substance.	

Test Preparation	3	Lock the Smart-Tip™ probe in the test unit. Press firmly.		STOP
		Flex Pins		

Identification	4	With thumb and forefinger , squeeze chamber 1 until the capsule breaks.		5	Gently shake the test unit for 2-3 seconds.	

Results		The appearance of a dark blue-green color indicates the presence of Ecstasy/MDMA or an opiate such as Heroin, Morphine or Codeine .	
		If no dark blue-green color develops, the above listed drugs are not present.	
		Proceed to Dilution (Steps 6-9), regardless of whether or not a color appeared.	

Dilution	6	With thumb and forefinger , squeeze chamber 2 until the capsule breaks.		7	Gently shake the test unit for 2-3 seconds.		The test window will warm up. This is normal.

Dilution	8	With thumb and forefinger , squeeze chamber 3 until the capsule breaks.		9	Gently shake the test unit for 2-3 seconds.		The test window will warm up. This is normal.

Disposal After Dilution, the test kit is safe to dispose of in the trash. ENG 07-22

IDenta
World Leader in Forensic Products

Manufactured by IDenta Ltd
www.identa-corp.com
Patented Technology
Made in Israel