



**Read Before Use**

As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

**Test Preparation**

**1**

Remove the Smart-Tip™ probe from test unit.




**2**

Sample suspected substance or surface.

**Substance & Trace Material**

Swipe the paper of the Smart-Tip™ probe across the suspect substance or surface.

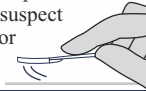

**Liquid**


Place the absorbent paper of Smart-Tip™ probe into liquid. Make sure paper is wet.



**Test Preparation**

**3**

Lock Smart-Tip™ probe into test unit. Press firmly.

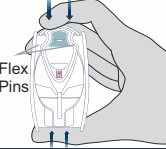


**STOP**  
Do not continue unless probe is locked below the Flex Pins and cannot be removed from the test unit.

**Chamber 1 Identification**


**4**

With **thumb and forefinger**, squeeze chamber 1 until capsule breaks.

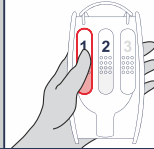

**5**

Gently shake test unit for 2-3 seconds.

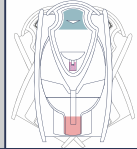

**Chamber 1 Results**



The appearance of turquoise color particles adhering to the sampler tip, the paper, or precipitating out of solution in either the front or back window indicates the presence of **Cocaine**.

*Color may disappear with time. Note: The liquid will remain pink.*

\* **Optional For Substance Testing Only:** If desired, confirm positive results with Cocaine/Crack Confirmation Kit (IDT1100).




If a turquoise develops, proceed to Disposal.  
If no turquoise color develops, proceed to Step 6.

**Chamber 2 Identification**


**6**

With **thumb and forefinger**, squeeze chamber 2 until capsule breaks.

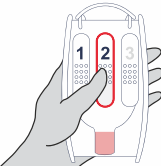

**7**

Gently shake test unit for 2-3 seconds.

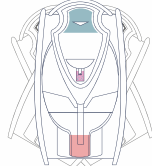

**Chamber 2 Results**



The appearance of turquoise color particles adhering to the sampler tip, the paper, or precipitating out of solution in either the front or back window indicates the presence of **Crack (Cocaine Base)**.

*Color may disappear with time. Note: The liquid will remain pink.*

\* **Optional For Substance Testing Only:** If desired, confirm positive results with Cocaine/Crack Confirmation Kit (IDT1100).



If no turquoise color develops, neither Cocaine nor Crack are present.



**Disposal**

The test kit is safe to dispose of in the trash.