


Read Before Use

As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

Test Preparation

1 Remove the Smart-Tip™ probe from test unit.

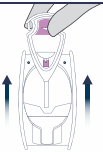


2 Sample suspected substance or surface. **Substance & Trace Material**

Swipe the paper of the Smart-Tip™ probe across the suspect substance or surface.



3 Lock Smart-Tip™ probe into test unit. Press firmly.

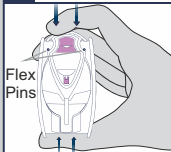


STOP

Do not continue unless probe is locked below the Flex Pins and cannot be removed from the test unit.

Identification

4 With **thumb and forefinger**, squeeze chamber 1 until capsule breaks.

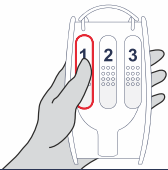

5 Gently shake test unit for 2-3 seconds.



Results



The appearance of a purple color in either the front or back window indicates the presence of an opiate, such as **Heroin**.



If no purple color develops, Heroin is **not** present.




Proceed to Dilution (steps 6-9), regardless of whether or not a color appeared.



Dilution

6 With **thumb and forefinger**, squeeze chamber 2 until capsule breaks.

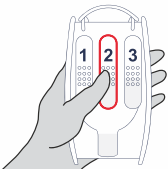

7 Gently shake test unit for 2-3 seconds.




Wait 5 minutes.
The test window will warm up. This is normal.

Dilution

8 With **thumb and forefinger**, squeeze chamber 3 until capsule breaks.



9 Gently shake test unit for 2-3 seconds.

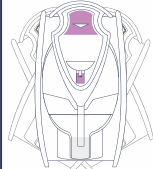

The test window will warm up. This is normal.



Disposal

After Dilution, the test kit is safe to dispose of in the trash.