



TESTING KITS FOR DRUGS

Procedure Card for IDT5500

AMPHETAMINE/ METHAMPHETAMINE CONFIRMATION

Substance Test



**Read
Before
Use**

As a precaution, gloves and eye protection should be worn when handling drug substances and using test units. Do not puncture any part of unit. Do not remove probe once a capsule is broken. In the unlikely event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.



This test is designed to enhance positive results from **Amphetamine/Methamphetamine Identification Kit (IDT5000)**. Only proceed to use this test after obtaining an orange result from the Identification Kit.

Test Preparation

1 Remove the Smart-Tip™ probe from test unit.



2 Sample suspected substance.

Powder

Touch Smart-Tip™ probe to sample.




Tablet

Firmly press Smart-Tip™ probe on the tablet while scrubbing in a circular motion until the tip is completely covered with substance.



Test Preparation

3 Lock Smart-Tip™ probe into test unit. Press firmly.

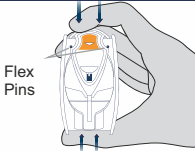


Flex Pins

STOP
Do not continue unless probe is locked below the Flex Pins and cannot be removed from the test unit.

Confirmation

4 With **thumb and forefinger**, squeeze chamber 1 until capsule breaks.

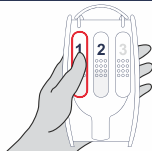


Confirmation

5 Gently shake test unit for 2-3 seconds.



6 With **thumb and forefinger**, squeeze chamber 2 until capsule breaks.

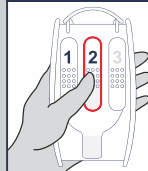

Do NOT shake or agitate test unit.

Results



The appearance of a blue color indicates the presence of **Methamphetamine**, if a positive result was received with Amphetamine/Methamphetamine Identification Kit (IDT5000).



If after a minute no blue color has appeared or a pink color appears, **and** if a **positive** result was received with Amphetamine/Methamphetamine Identification Kit (IDT5000), the presence of **Amphetamine** is indicated.



Disposal

The test kit is safe to dispose of in the trash.

01-19



World Leader in Forensic Products

Manufactured by IDenta Ltd

www.identa-corp.com

Patented Technology

Made in Israel