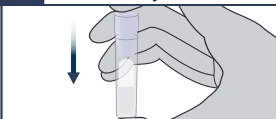


SWIPPER™**DISCREET
DRUG TEST****Synthetic
Cannabinoids
Test Kits****SW-3400****Read
Before
Use**

As a precaution, gloves and eye protection should be worn when handling substances and using test units. Do not puncture any part of unit. Do not remove probe after breaking the ampule. In event of a spill, rinse affected area thoroughly with water. Keep out of reach of children. Store the tubes in a dark place, not exposed to light. All results are presumptive. If necessary, further testing by a competent drug testing lab can provide additional identification.

Test Preparation**1****Remove the
SWIPPER™ Cap
from the test unit.**

Ensure that the
test tube remains
upright in order
to avoid the
ampule falling
out of the tube.

2**Sample the suspected
substance with the
SWIPPER™ Cap.****Test Preparation****3****Return the SWIPPER™
Cap to the test tube.
Press firmly.****STOP****Do not continue unless
the SWIPPER™ Cap
is properly secured.****⚠ CAUTION ⚠****The kit contains an acid.
Make sure to follow all
the safety procedures.****Identification****4****With thumb and forefinger,
squeeze the test tube
until the ampule breaks.****5****Gently shake test up and
down to ensure that the
liquid touches the Cap.****Results**

The appearance of a yellow color indicates
the presence of a **Synthetic Cannabinoid**
(street names – **K2, Spice**), such as:

Naphthoylindoles: JWH - 007, 011, 015, 016, **018**, 019, 020, **022**,
072, **073**, **081**, 098, **122**, **133**, 180, 182, 200, **210**, 213, 398, 424.

Benzoylindoles: AM - 223, **694**, **RCS - 4**.

Nathoylpyrroles: JWH - 030, 031, 145, 147, 307, 309, 368, 369, 370.



If **no** yellow color develops,
the above listed drugs are **not** present.

* The test will **not** react to the following substances:
Classical cannabinoids, nonclassical cannabinoids,
phenylacetylindoles.

**Disposal**

Dilute the acid with some tap water
before disposing the kit in the trash.

ENG
v1
01-22**World Leader in
Forensic Products**

Manufactured by IDenta Ltd
www.identa-corp.com

Patented Technology • Made in Israel

LOT:**EXP:**