enUS	Instructions for use  3
fr	Notice d'utilisation  18
es	Instrucciones de uso  33

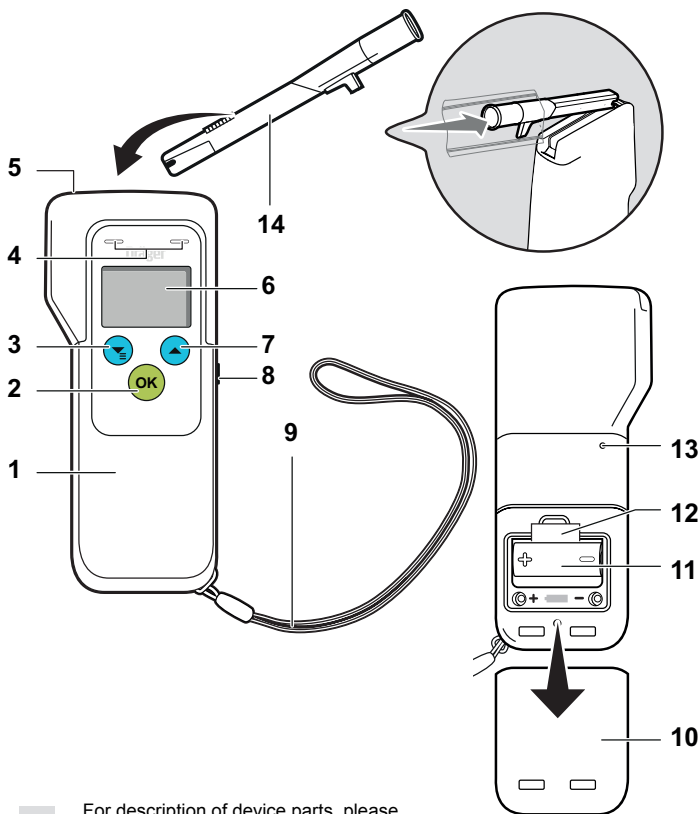
Dräger Alcotest 5820

WARNING

Strictly follow the Instructions for Use.
The user must fully understand and strictly observe the instructions.
Use the product only for the purposes specified in the Intended use section of this document.

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1 For your safety

1.1 General safety notes

- Before using this product, carefully read the Instructions for Use.
- Strictly follow the Instructions for Use. The user must fully understand and strictly observe the instructions. Use the product only for the purposes specified in the Intended use section of this document.
- Do not dispose of the Instructions for Use. Ensure that it is retained and appropriately used by the product user.
- Comply with all local and national rules and regulations associated with this product.
- Only trained and competent personnel are permitted to inspect, repair and service the product. Dräger recommend a Dräger service contract for all maintenance activities and that all repairs are carried out by Dräger.
- Do not use a faulty or incomplete product. Do not modify the product.
- Notify Dräger in the event of any component fault or failure.

2 Conventions used in this document

2.1 Definitions of alert icons

The following alert icons are used in this document to provide and highlight areas of the associated text that require a greater awareness by the user. A definition of the meaning of each icon is as follows:

WARNING

Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

2.2 Typographical conventions

 This symbol indicates information which facilitates the use of the product.

3 Description

3.1 Product overview

- 1 Breath-alcohol-analyzer
- 2  button (switch instrument on/off; confirm inputs)
- 3  button (open menu; set value)
- 4 LEDs
- 5 Mouthpiece receptacle (configurable: right or left)
- 6 Display
- 7  button (set value)
- 8 USB connection (Important: Not for charging the device, for Service only)
- 9 Wrist strap
- 10 Battery compartment cover
- 11 Battery
- 12 Pull strap for battery replacement
- 13 Loudspeaker (Important: Not reset button)
- 14 Mouthpiece

3.2 Intended use

The Dräger Alcotest[®] 5820¹⁾ is used for measuring the alcohol concentration in the breath of a human. The measurement is used as an indication for alcohol intoxication. Based on the measurement results no therapeutic measures should be initiated.

1) Alcotest is a registered trademark of Dräger.

4 Use

WARNING

Small parts of the device can come loose and be swallowed.

- ▶ Keep the device out of reach of children under the age of 3.

4.1 Prerequisites

The instrument is factory-calibrated when delivered. If the calibration date has exceeded 12 months, the instrument should be adjusted, e.g. by DrägerService. It is still possible to use the device, even if the calibration period is exceeded. After the last alcohol intake, wait approx. 15 minutes before a measurement is carried out. Residual mouth-alcohol may cause incorrect measuring results. This may also occur by aromatic drinks (e.g. fruit juices), alcoholic mouth sprays, medicines and drops, as well as by burping and vomiting. Rinsing the mouth with water or non-alcoholic drinks does not replace the wait time!

4.2 Initial start-up

A quick setup process (“wizard”) starts when the instrument is switched on for the first time. This step is not required for pre-configured instruments.

The following settings are accessible using the wizard:

- Language
- Date format
- Date
- Time format
- Time
- Measuring unit

All settings can be changed later on via the menu.

4.3 Carrying out an automatic breath test

A new mouthpiece must be inserted for each new person who provides a breath test.

Do not allow any high alcohol concentrations on the sensor, e.g. by rinsing the mouth with a high proof alcohol shortly before the measurement, as this will shorten the service life of the sensor.

1. Remove mouthpiece (order no. 68 10 690 “Standard”, 68 11 055 “with back valve”)
2. For reasons of hygiene, leave the protective wrapping around the mouth contact area. Open the packaging at the perforation and take off the protective wrapping only around the attachment area of the mouthpiece.
3. Attach the mouthpiece at an angle to the mouthpiece receptacle.
4. Press it into the receptacle. The fluted markings (press-on surface) on the mouthpiece faces upwards while this is done. The mouthpiece audibly snaps into place (see page 2).
5. Remove all packaging.
6. Keep the  button pressed for approx. 1 second until the startup screen is displayed. An optional self-test is carried out.
7. After approx. 4 seconds the display shows **READY** and the instrument is now ready to test. Take a deep breath and blow evenly and without interruptions into the mouthpiece as long as possible. A progress bar is displayed. An adequate breath flow is confirmed by a continuous tone.
8. Without any further action from user side, the device takes automatically a test sample at the end of the breath of the test subject.
9. The continuous tone is muted directly afterwards which gives the signal that the test subject can stop with blowing. The test result is displayed with date and time after 5 to 25 seconds (depending on the instrument temperature and the measured concentration).

In case of a negative alcohol test, **0,00** will be displayed.

In case of a positive alcohol test, the measured alcohol level will be displayed.

To perform a new measurement, press the  button.

After the sensor's short recovery time, (see "Technical data" on page 15) **READY** will be displayed and the device is ready to use again.

4.4 Performing a manual breath test (optional)

If the test subject cannot provide the required minimum volume, the sampling of breath can be triggered manually.

1. Preparation as for the automatic breath test (see chapter 4.3 on page 6).
2. If the progress bar is displayed after breath flow is detected, tap the  button.

In the results display, **MANUAL** is also displayed.

4.5 Performing a passive breath test (optional)

With passive sampling, the sampling is initiated manually by the instrument user. The passive test must be done without an attached mouthpiece. Possible applications include the testing of ambient air or the breath of persons for the presence of alcohol. To perform a passive test:

1. Keep the  button pressed for approx. 1 second until the startup screen appears.
2. Hold the instrument into the air to be analyzed **without** a mouthpiece.
3. Briefly press the  button to initiate the measurement.
The device will take a test sample at that very moment and will start analyzing.

In the results display, **PASSIVE** is also displayed. If no alcohol is detected, the **No Alcohol** will be displayed and the LED will be green. In case alcohol is detected, **Alcohol** will be displayed and the LED will be red.

4.6 Switching off the Instrument

The instrument will switch off after pressing the button for approx. 3 seconds. A switch-off count down will be shown as well. If the display is off, stop pressing the button. Otherwise the device will be switched on again.

The instrument will switch off automatically after approx. 4 minutes without any activity.

Switching off the instrument does not reduce the post-test waiting period before you can perform another test.

4.7 Removing the mouthpiece

1. Push the mouthpiece upwards out of the mouthpiece receptacle.
2. Dispose of the mouthpiece in accordance with local regulations.

Do not reuse a mouthpiece.

4.8 Settings/information

Start the device pressing until **READY** is displayed.

To enter the device menu, press the .

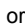
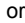
In the main menu all menu options are listed one below the other.

To navigate within the options list and select a menu option:

1. Press or button to mark the respective menu option with the cursor.
2. Confirm the selected menu option with the .

The following table provides an overview of the menu options and how to use them:

Menu option	Description	Interaction instruction
Measuring	Option to leave the menu and to go back to the READY screen to perform a test.	<ul style="list-style-type: none"> press OK to leave the menu and prepare for measuring
Results	Overview of the last tests performed on the device (incl. result, date, time, test ID). There is one test displayed per screen.	<ul style="list-style-type: none"> press OK to enter the menu press ↓ or ↑ to navigate through the list
Date/Time	Date: to set the date Time: to set the time Date format: to choose a format (yyyy/mm/dd, yyyy-mm-dd, mm/dd/yyyy, dd.mm.yyyy) Time format: to choose a format (12 or 24)	<ul style="list-style-type: none"> press OK to enter the menu press ↓ or ↑ to navigate through the list press OK to select the option change a setting or number by pressing ↓ or ↑ press OK to confirm, the submenu will be displayed again select Back to leave the submenu

Menu option	Description	Interaction instruction
Contrast	The display contrast can be adjusted within 5 levels.	<ul style="list-style-type: none">● press  to enter the menu● change the setting by pressing  or ● press  to confirm
Speaker	Any acoustic signals of the device can be turned off or to a volume of 50% or 100%.	<ul style="list-style-type: none">● press  to enter the menu● change the setting by pressing  or 
Language	The language of the display can be adjusted.	<ul style="list-style-type: none">● press  to enter the menu● choose the language by pressing  or ● press  to confirm
Meas. unit	The measurement results can be displayed in the following units: %, ‰, g/L, mg/100 mL, mg%, mg/mL, mg/L, µg/100 mL, µg%, µg/L, g/210 L.	<ul style="list-style-type: none">● press  to enter the menu● choose the unit by pressing  or ● press  to confirm
Software Version	Software version and part number are displayed.	<ul style="list-style-type: none">● press  to enter or leave the menu

Menu option	Description	Interaction instruction
Login	Option for service technicians only. Do not enter this submenu.	
Switch off	Device can be switched off.	<ul style="list-style-type: none">● Hold the button for 4 seconds to switch off the device.

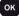
An error message is displayed in the event of an instrument fault. The number that appears below the message is used for service functions. If the fault remains after switching the instrument on and off several times, contact DrägerService.

5 Troubleshooting

Fault / message	Cause	Remedy
Instrument cannot be switched on.	Battery empty.	Replace the battery.
Volume too low / sample invalid.	Test subject is blowing intermittently or not hard enough.	Blow more strongly and without interruptions into the mouthpiece.
Volume too large	Measuring range exceeded.	Repeat breath test.
Instrument outside temperature range	Instrument too hot or too cold.	Allow the instrument to cool down or become warmer.
Measuring range exceeded	Measuring range exceeded. Residual mouth alcohol may cause incorrect measuring values.	There must be an interval of at least 15 minutes after alcohol has been taken into the mouth (see page 6). To repeat, press the  button.
Sample interruption	Abrupt breath interruption or inhalation detected	Repeat breath test.
Fault number	Instrument fault	Switch the instrument off and on again. If the fault remains, contact DrägerService.

6 Maintenance

Dräger recommends a device calibration every 12 month, because of the sensor drift the uncertainty of measurement increases over time. Instrument calibration and maintenance should only be carried out by authorized persons, e.g. DrägerService.

The last 30 days before next adjustment / service is needed, a day count down will be shown as information in the display after switching on the device. You have to confirm this information with the  button. After the adjustment / service date the device starts to count up how many days it is over requested adjustment / service. This information has to be confirmed by pressing the  button as well. It shall remember you that the device needs to be adjusted / serviced.

Have the instrument re-calibrated, if an accuracy check does not produce a result per NHTSA's EBT Model Specifications. Calibrations should only be carried out by authorized persons, e.g. DrägerService.

6.1 Replacing the battery

1. If necessary, switch off the device.
2. Slide the battery cover off.
3. Replace the battery (battery type: 3 V CR123A; order no. 83 21 390), by observing the correct polarity.
4. Replace the battery cover.
5. Verify that the time is set correctly.

If the battery replacement process takes longer than 5 minutes, the time must be reset. A measurement is only possible with the time set.

6.2 Cleaning

Do not immerse in fluids and avoid contact of fluids with the electrical receptacles. Do not use any cleaning agents containing alcohol, solvents or substances containing citrus. Wipe down the instrument with a disposable cloth.

7 Technical data

Measuring principle

Electrochemical sensor

Measuring range

Breath alcohol concentration 0.000 to 0.500 g/210L

Blood alcohol concentration 0.000 to 0.500% BAC

Small values will be shown as 0.000 to 0.0063 g/210L
0.000

Ethanol mass per blood volume or mass at 20°C/ 68 °F and 1013 hPa.
For the above data, a conversion factor of 2100 is used for breath
and blood alcohol concentration data.

Units of measure

%, ‰, g/L, mg/100 mL, mg%, mg/mL, mg/L, µg/100 mL, µg%, µg/L, g/210 L

Ambient conditions

During operation -5°C to +50°C / +23°F to +122°F
10 to 100% rel. hum.
(non-condensing)
600 to 1300hPa

During storage -20°C to +70°C / -4 to +158°F

Taking Samples

Respiratory volume >1.3 L (default)

Minimum flow rate >10 L/min (default)

Minimum blow time >1.6 s (default)

Measurement accuracy

Standard deviation of the repeatability with ethyl alcohol standard

Up to 0.50 mg/L	0.008 mg/L
≥0.50 mg/L	1.7% of the measured value
up to 0.100 % BAC	0.0017 % BAC or
≥ 0.100 % BAC	1.7% of the measured value whichever value is higher

Sensitivity drift

Typically 0.4 % of the measured value per month

Wait time for measurement readiness following a previous measurement with:

0.25 mg/L	20 s	0.050 % BAC	20 s
0.50 mg/L	40 s	0.100 % BAC	40 s
1.00 mg/L	80 s	0.200 % BAC	80 s
>1.50 mg/L	120 s	>0.300 % BAC	120 s

Calibration

every 12 months

Dimensions

1.96/2.36 " x 5.55" x 1.22" (W x H x D)

Weight

approx. 144 g/ 5.1 oz

Power supply

1 x 3 V CR123A

Interface

USB, 2.0 fullspeed (for service only)

Protection class

IP 54

CE markings

Electromagnetic compatibility (Directive 2014/30/EU).

Configuration possible, see name plate

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

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© Dräger Safety AG & Co. KGaA
Dräger Safety AG & Co. KGaA
Dräger, Inc.